medela EN

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Material: Breastshield and Connector (Polypropylene), Valve Membrane (Thermoplastic elastomer), Tubing (Polyvinyl chloride).

Disposal: According to local regulations.

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Instructions for use Symphomy and Lactina One-Day Pump Set

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Important Safeguards

- This is a single user product.** Use by more than one person may present a health risk as the components cannot be sterilised for safe re-use between multiple users.
- Follow the cleaning instructions unless you are told otherwise by your healthcare professional.
- Do not use when package is opened or damaged prior to first use.
- Use Medela original accessories only.
- Never leave accessories unattended and keep out of reach of children.
- If problems or pain occur, consult your lactation consultant or physician.
- For further information refer to Symphony or Lactina instructions for use.

Application

- Choose the right breastshield size for maximum comfort and efficiency while pumping. For breastshield fitting guidelines refer to Symphony or Lactina instructions for use.
- Use a disposable breastmilk bottle according to the individual needs (various bottle sizes available).
- Additionally for use with Symphony, Symphony Membrane Cap and Protective Membrane. For use with Lactina, Lactina Flange Cylinder and/or, Cylinder Holder. We recommend to exchange/clean these extra parts regularly or in case of an overflow, immediately.
- Assemble all parts, connect with breastpump and start pumping.

Important

- A new Symphony and Lactina One-Day Pump Set is required:
 - for each mother,
 - after a maximum of 8 pumping sessions within 24 hours
 - every 24 hours
 - if milk accidentally gets into the tubing
- A new disposable breastmilk bottle is required:
 - for each pumping session

Assembly/Disassembly instructions

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- Use drinking water quality for cleaning.
- Disassemble all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

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Caution

- This Symphony and Lactina One-Day Pump Set is not autoclavable.
- Use of other sanitizing methods (e.g. cold water disinfection) is not recommended and may damage some parts.

Prior to first use

The Symphony and Lactina One-Day Pump Set does not need cleaning prior to first use as it is provided sterile.

- Check what there is milk in the tubing. If so, replace the tube with clean water and basic disinfecting liquid (approx. 30°C). Rise with cold clear water. Note that the other parts do not need cleaning.
- Wipe the parts with a clean towel. Put all parts in a clean Storage bag (not airbag) or in a clean environment.

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