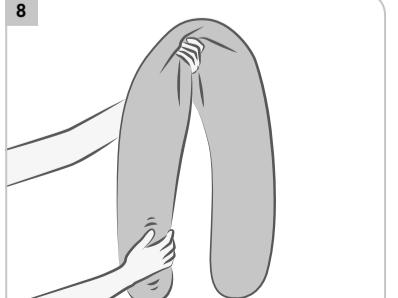
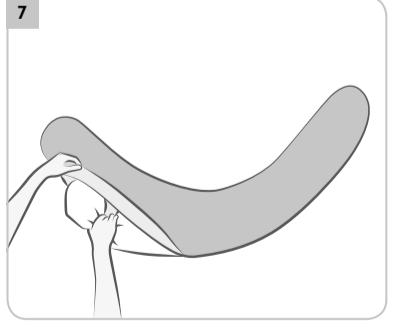
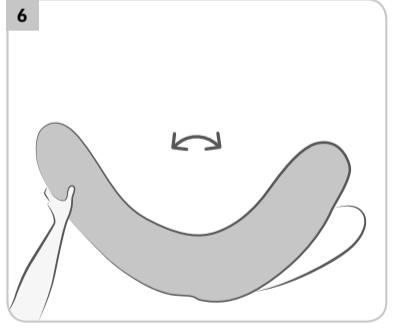
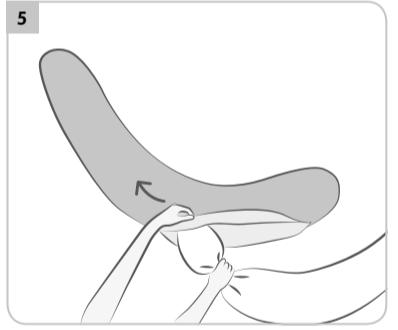
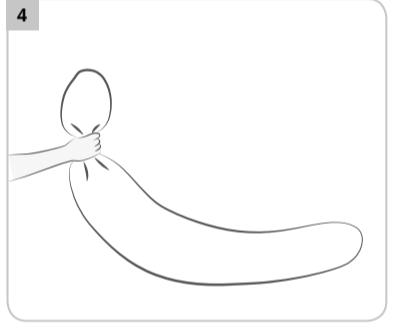
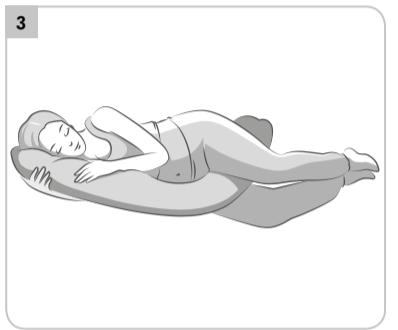
**Instructions for use  
Maternity and nursing pillow****Gebrauchsanweisung  
Schwangerschafts- und Stillkissen****Mode d'emploi  
Coussin de maternité et d'allaitement****Istruzioni per l'uso  
Cuscino per la gravidanza  
e l'allattamento****Gebruiksaanwijzing  
Zwangerschaps- en voedingskussen****Instrucciones de uso  
Cojín de embarazo y lactancia****Instruções de uso  
Almofada de Gravidez e Amamentação****Bruksanvisning  
Gravidets- och amningskudde****Bruksanvisning  
Gravidets- og ammepute****Instrukcja obsługi  
Poduszka dla kobiet w ciąży  
i karmiących piersią****Инструкция по применению  
Подушка для беременных  
и кормящих мам**

**Medela AG**  
Lätičhstrasse 4b  
6341 Baar, Switzerland  
www.medela.com

**International Sales**  
Medela AG  
Lätičhstrasse 4b  
6341 Baar, Switzerland  
Phone +41 41 562 51 51  
www.medela.com

© Medela AG/ref. medier doc. 101036152/A

EN

**Intended use**

The Medela maternity and nursing pillow is developed to support mothers and their bodies during their nursing (Fig. 1), pumping breast milk (Fig. 2) and relaxing (Fig. 3) moments, at day and night time. Always use the Medela maternity and nursing pillow with a pillow cover on, as this protects the pillow insert from damage.

**Important safety information**

**DANGER – Risk of suffocation (due to packaging)**  
Plastic bags can become a deadly trap when being played with or crawled into.

• Keep the packaging away from children and animals.

**WARNING – Risk of suffocation (due to the use of the pillow)**  
Leaking filling material (micro pearls) can harm the respiratory tract. Children could breathe in or swallow leaking micro pearls.

• Use your Medela maternity and nursing pillow only with a pillow cover on.

• Before using the pillow, inspect it for defective spots and leaking micro pearls.

• Since micro pearls are not edible, make sure that your baby does not eat any micro pearls.  
Each pillow may pose a suffocation risk for infants (for example, pillows can slip over the face during sleep).

• Never leave your baby unattended with the Medela maternity and nursing pillow.

**NOTICE – In the following cases some colour change to the product or its packaging may occur:**

- It is new or possesses a strong colour, or parts of the product got wet.
- It was exposed to direct sunlight for a longer period of time.

These colour changes do not limit the product usability.

**Cleaning and hygiene**

Before washing the pillow insert and its cover, please read and follow the instructions on the separate care labels attached.

Additionally, check the pillow insert carefully for leaks or seam defects before washing.

**Wash instructions**

1. Whenever possible, wash the pillow insert and the pillow cover separately in a laundry bag.

2. Turn the pillow cover inside out and wash with closed zip (leave about 10 cm open).

3. Wash the pillow cover at 40 °C and the pillow insert at 30 °C.

4. Dry both the pillow insert and the pillow cover. Tumble dry only at the most gentle level and with sufficient space for the pillow insert to move freely inside the dryer.

Do not wash or dry the pillow parts with other coloured clothes.

**NOTICE – Wash and tumble dry only in machines large enough to allow the pillow to move freely. Leaking micro pearls can damage your machine.**

Recommendation:  
If you do not have a sufficiently large laundry bag at hand, you can also use the pillow cover for this.

In this case the wash temperature is limited to 30 °C.

**NOTICE – With new pillow covers and strong colours, a colour change to the pillow insert may occur when parts are washed together.**  
However, this does not limit the product usability.

**NOTICE – To prevent damage to the pillow insert, which results in leakage of micro pearls, always wash and dry it inside a laundry bag, or use the pillow cover as a protection. Failure to do so voids the warranty of the product.**

**Disinfection**

If you want to disinfect the pillow, please use only spray disinfectant. The pillow insert and the pillow covers are treatable with all conventional disinfectants.

**Putting the pillow cover on**

1. Hold up a 15 to 20 cm long "nose" at the top end of the pillow insert [Fig. 4]. This causes the filling to drop into the lower part of the pillow insert.

2. Push the "nose" upwards into the very end of the pillow cover [Fig. 5].

3. Turn the pillow while holding the pillow and the cover from the outside together with one hand [Fig. 6].

4. After the filling has dropped downwards, grab a new "nose" on the other side and push it all the way into the pillow cover's other end [Fig. 7].

5. Distribute the filling evenly in both ends of the pillow [Fig. 8].

6. Close the zip of the pillow cover.

**Refilling micro pearls**

Due to the characteristics of the micro pearls and depending on the type and duration of the use, the volume of the filling may shrink.

If you like to increase the filling volume, you can purchase a refill pack and proceed as follows:

1. Remove the pillow cover from the pillow insert.

2. Find the refill opening and open to make the refill pocket accessible.

3. Open the zip of the refill pocket.

4. Put the tube of the refill pack into the opening and allow the micro pearls to flow into the pillow.

5. When the refilling is done, remove the refill pack from the opening.

6. Close the refill pocket.

**WARNING – Health hazards and material damage due to leakage of micro pearls (see "Important safety information").**

To keep the Medela maternity and nursing pillow safe and apt for its intended use always close the refill opening in the pillow insert properly before using the pillow again.

**Product specifications**

Dimension: 170 x 32 cm (67 x 12.6")

Material of pillow cover: 100 % pure cotton

Material of pillow insert: Non-woven fabric made of 100 % polypropylene

Filling material: EPS (expanded polystyrene) micro pearls

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