

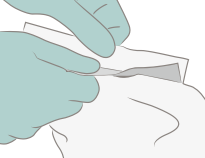
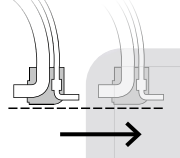
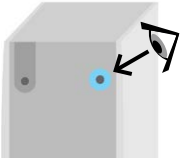


# Thopaz™ Quick Card for OR

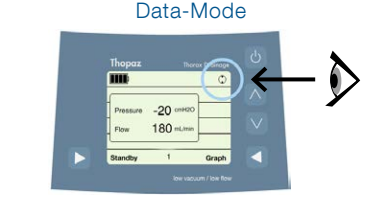
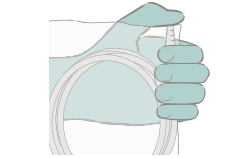
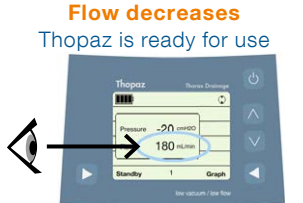

## Prepare Thopaz for use

 <p>single</p> <p>double</p> <p>select tubing</p>	 <p>0.3L    0.8L    2.0L</p> <p>select canister (option: with solidifier)</p>	 <p>open sterile packaging</p>	 <p>attach tubing</p>	 <p>check presence of sealing ring</p>
--	--	--	--	---

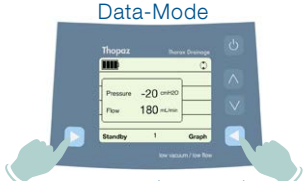
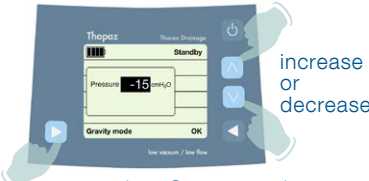

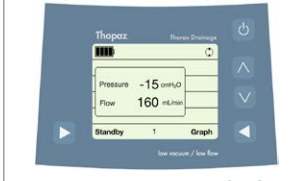
## Turn Thopaz on

 <p>attach canister</p>	 <p>ready for use</p>	 <p>press power button</p>	<p>Choose patient therapy number</p>  <p>keep number    new number</p>	<p>Standby-Mode</p>  <p>therapy number    switch on pressure</p>
--	--	---	--	---

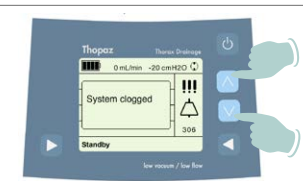

## Perform functional check

 <p>Data-Mode</p> <p>check Thopaz is running</p>	 <p>seal off patient tubing (keep sterile)</p>	<p>Flow decreases Thopaz is ready for use</p>  <p>check flow rate</p>	 <p>connect patient according to hospital guidelines</p>
---	---	---	---

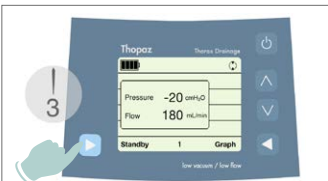

## Adjust pressure (when pump is running)

 <p>Data-Mode</p> <p>press simultaneously</p>	 <p>increase or decrease</p> <p>or select Gravity mode</p>	 <p>confirm with OK</p>	 <p>new setting is applied</p>
--	---	---	---

## Handle alarms

<p>Alarm</p>  <p>press simultaneously</p>	<p>Troubleshooting</p>  <p>follow advice on screen</p>
--	---

## Switch off Thopaz

<p>Switch to Standby</p>  <p>press longer than 3 seconds</p>	<p>Switch off</p>  <p>press once quickly</p>
--	---

 Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
www.medela.com

**International Sales**  
Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
Phone +41 41 562 51 51  
Fax +41 41 562 51 00  
ism@medela.ch  
www.medela.com

**CAUTION:**  
Federal US law restricts this device to sale by or on the order of a physician.

**This Quick Card in no way replaces the Instructions for Use REF 200.0685**

 0123

Precious life – Progressive care